



FilitaNeb™

Cirrus™2 nebuliser, adult, Intersurgical EcoLite™ non-vented mask kit with filter and tube



Oxygen and Aerosol Therapy ▪ Nebuliser Therapy

To meet the changing requirements of the global healthcare market this product option has been developed to reduce particulates entering the environment during nebuliser therapy. The FilitaNeb™ features a non-vented mask based upon the Intersurgical EcoLite™ mask, with a filter on the exhalation gas pathway. The mask seal is coloured dark green to distinguish this non-vented mask from standard aerosol masks. To ensure that the best possible fit can be achieved, the mask kit includes an additional head strap.

The system provides >95% reduction of particulates into the environment, helping to minimise potentially infected exhaled gases and emissions escaping during nebulisation therapy, reducing the risk to healthcare professionals.

The Cirrus™2 nebuliser incorporates a number of features, including a graduated drug cup that provides guidance on how much drug is in the nebuliser. The cup has a quick-release fitting which enables the nebuliser to be attached, as well as detached, with a quarter turn. This speeds up the filling process and provides a more reliable seal between the cup and the nebuliser body.

The Cirrus2 nebuliser is designed to deliver drugs for tracheobronchial deposition.

Incurved nose seal

Conforms to different nose shapes designed to reduce the risk of aerosolised drug entering patient's eyes

Filter on exhalation port

Reduces particulates entering the environment during nebuliser therapy

Graduated nebuliser cup guide

Faster preparation

Additional head strap

provides an improved seal between the mask and the patient's face

Soft face seals

Increased patient comfort

Dark green mask seal


Differentiates the non-vented mask from standard aerosol masks

A choice of "under chin" or "on chin" positions

Provides a better fit on a wider range of patient face shapes

Quick-release fitting

Attaches and detaches easily

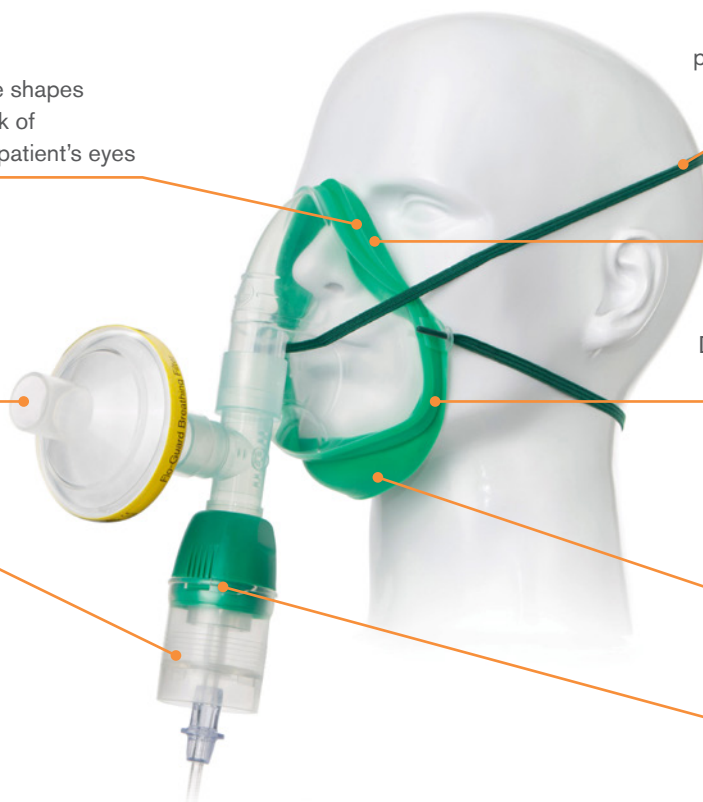

Code	Description	Tube length	Box Qty.
1453044	FilitaNeb™ Cirrus™2 nebuliser, adult, Intersurgical EcoLite™ non-vented mask kit with filter and tube	2.1 m	12

Lower environmental impact product

IS10.39 • Issue 5 12.24



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com



The manufacturer Intersurgical Ltd is certified to ISO 14001:2015, ISO 9001:2015, ISO 13485:2016 and MDSAP

Please think before you print
Save energy and paper.
If you must print this information sheet please print it double sided.